

Prior to installation, inspect the unit to ensure no tampering or damages were incurred.

1. Securely anchor the EOSC to the exterior wall using fasteners appropriate for the substrate. Installation fasteners are not provided with the Powerex EOSC.
2. Install appropriate pipe hangers on pipeline to prevent the weight of supply lines from being supported by the EOSC.
3. Braze the pipe connection to the pipeline system in accordance with NFPA 99.
4. NFPA 99 requires that all associated check valves be installed on the interior of the building. One 75 PSI relief valve with 1/2" NPT ports is included. Check valves are sold separately.
5. Apply caulking per the relevant specification, local building code, or the authority having jurisdiction.