


## Install Instructions

### Nitrogen Control Panel Conversion Kit for Oxequip Back Box OR Powerex Aluminum Back Box

Compatible with ¼ OD plastic tube connector risers

**CAUTION:** *Shut off the supply of nitrogen control panel to be converted before proceeding.*


1. Remove the original Nitrogen Control Panel front.
2. Disconnect the Nitrogen supply line and Nitrogen outlet line tubing at the risers.
3. Install the sub-plate with the large cutout to the original back box using the #6 x 1 inch flat head screws in the appropriate holes (circled holes will align with existing rough in). Screw heads should be flush with panel.
5. While holding the new Nitrogen Control Panel, push the black tubing firmly into the Nitrogen supply line fitting and the yellow tubing firmly into the Nitrogen outlet line fitting. Pull on tubing to ensure it is locked in place.

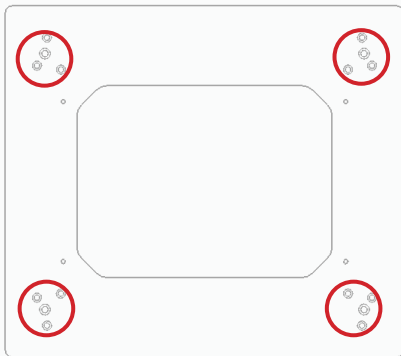

4. The tubing provided in this panel should be connected and disconnected by hand – without the use of tools. **Note:** Wash your hands before handling these connections to ensure that oil, grease, dirt etc. do not contaminate the fittings possibly causing leakage and introducing impurities into the medical gas system.

to connect


to disconnect

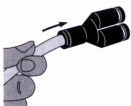

Simply push the tubing until it can go no further. Holding and sealing is accomplished instantaneously.



First depress the manual release button, then pull the tubing out of the fitting.

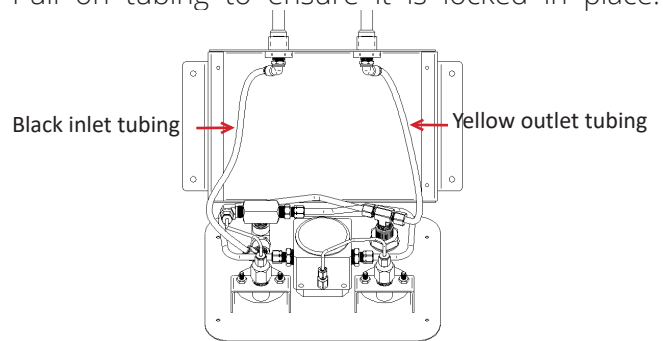

6. Secure the new Nitrogen Control Panel to the sub-plate with the four short #6 pan head screws in the positions shown below.

