


FIELD FURNISHED
DISCONNECT IS
REQUIRED

SEE NOTES ON
DATA TABLE PAGE

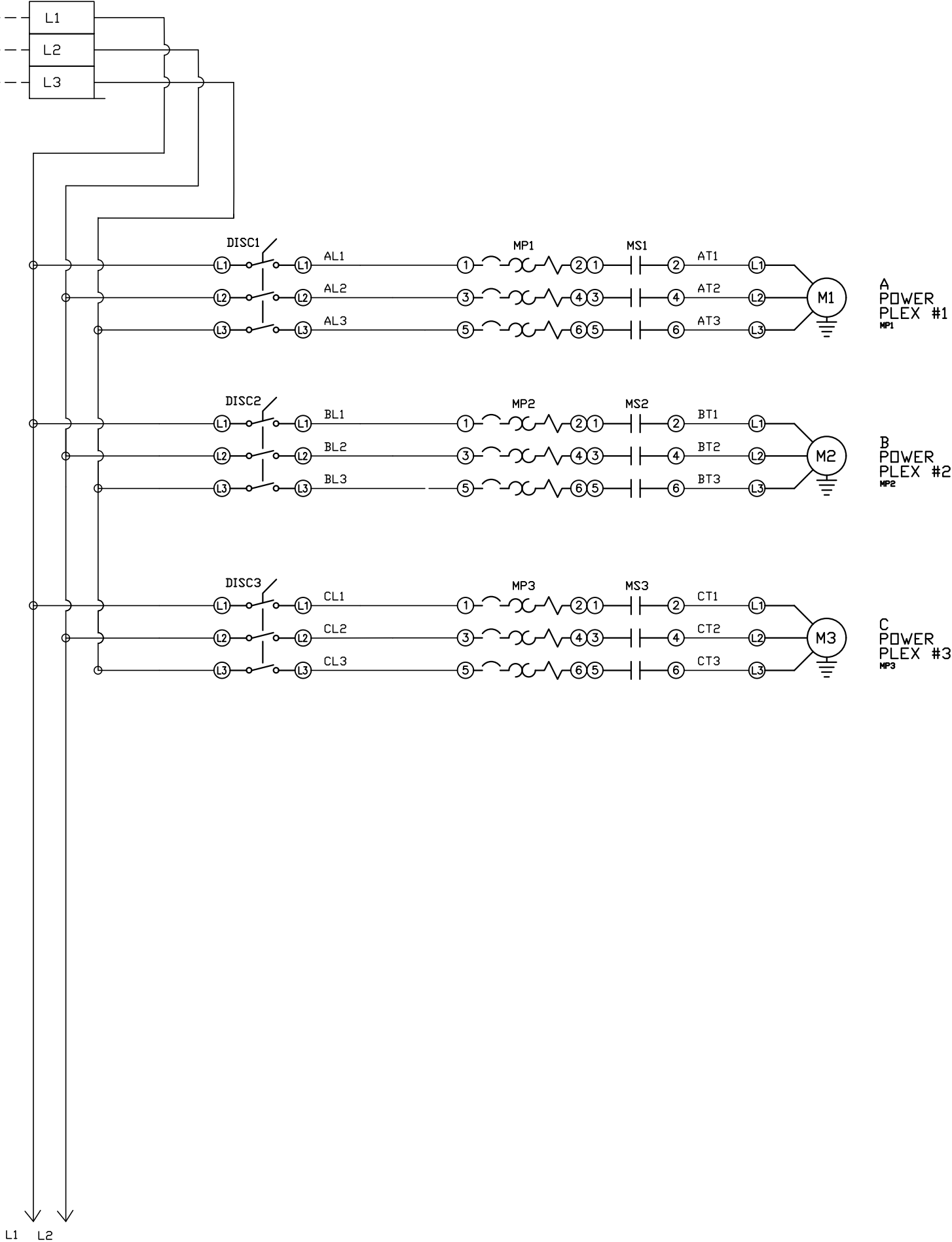

REV	REVISION	DATE	ECN	NAME	CHKD
B-1	RELEASE TO PRODUCTION	07/20/11	22545	CHR	CHR
G-1	ADDED GROUND TERMINAL BLOCKS	04/11/25	PXEC0688	JNW	JRD
H-1	ADDED PLEX ISOLATION	05/29/25	PXEC0639	SEK	WMM

OPTIONAL:

POWEREX™

CONFIDENTIAL DISCLOSURE:
This drawing is the property of POWEREX, INC. of Marmon Specialty Products and subject to return on demand. Its contents are confidential and must not be copied or submitted to outside parties for use or examination.

DRAWN BY	CHECKED BY	ENGINEERING APPROVAL
CHR	BFH	DMS
07/20/11	03/12/12	03/12/12


PANEL TYPE
MED AIR TRIPLEX
HMI, NFPA

DWG. TYPE WIRING

DWG. NO.
PXMI-A316 W

SHEET	SIZE
W-1	B

REV	REVISION	DATE	ECN	NAME	CHKD
B-1	RELEASE TO PRODUCTION	07/20/11	22545	CHR	CHR
G-1	ADDED GROUND TERMINAL BLOCKS	04/11/25	PXEC0688	JNW	JRD
H-1	ADDED PLEX ISOLATION	05/29/25	PXEC0639	SEK	WMM

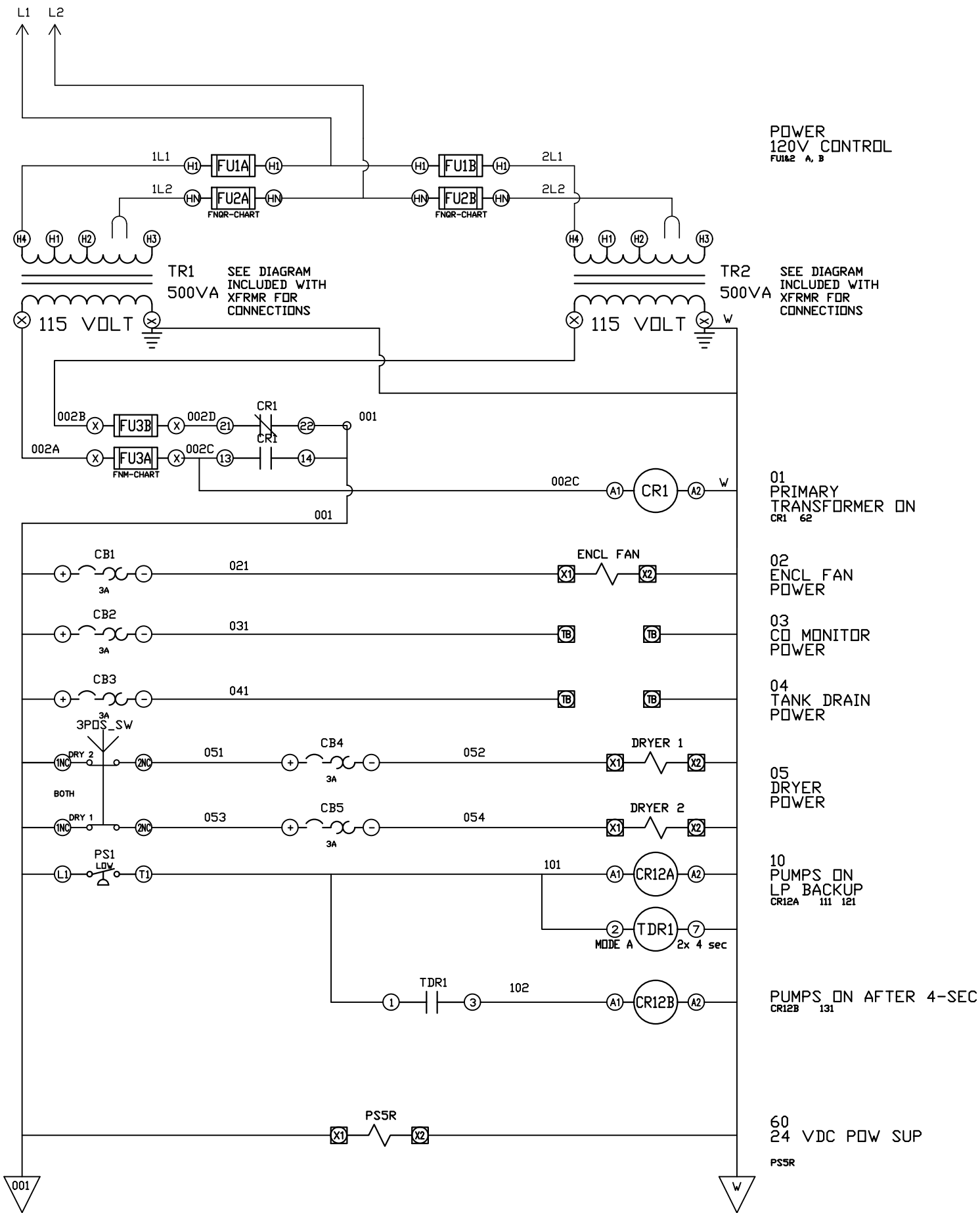

OPTIONAL:

POWEREX™

CONFIDENTIAL DISCLOSURE:
This drawing is the property of POWEREX, INC. of Marmon Specialty Products and subject to return on demand. Its contents are confidential and must not be copied or submitted to outside parties for use or examination.

DRAWN BY	CHECKED BY	ENGINEERING APPROVAL
CHR	BFH	DMS
07/20/11	03/12/12	03/12/12



PANEL TYPE
MED AIR TRIPLEX
HMI, NFPA

DWG. TYPE WIRING

DWG. NO.
PXMI-A316 W

SHEET	SIZE
W-2	B

REV	REVISION	DATE	ECN	NAME	CHKD
B-1	RELEASE TO PRODUCTION	07/20/11	22545	CHR	CHR
G-1	ADDED GROUND TERMINAL BLOCKS	04/11/25	PXEC0688	JNW	JRD
H-1	ADDED PLEX ISOLATION	05/29/25	PXEC0639	SEK	WMM

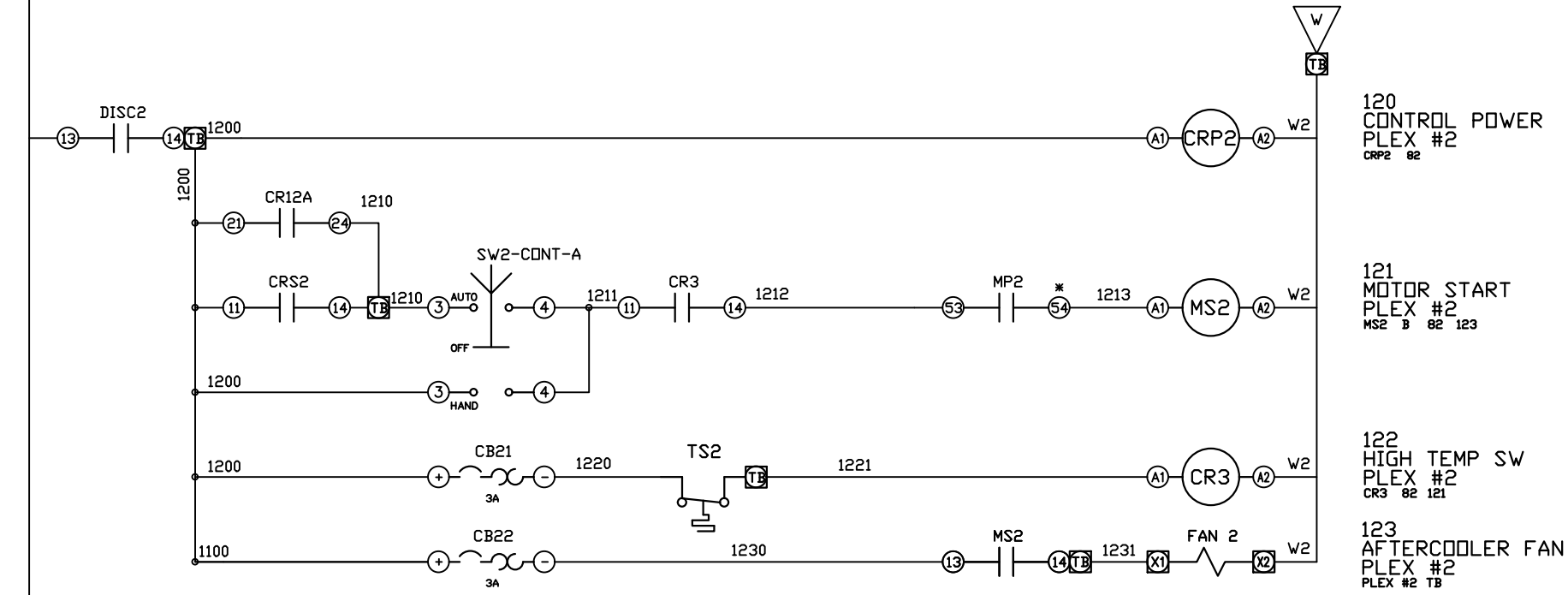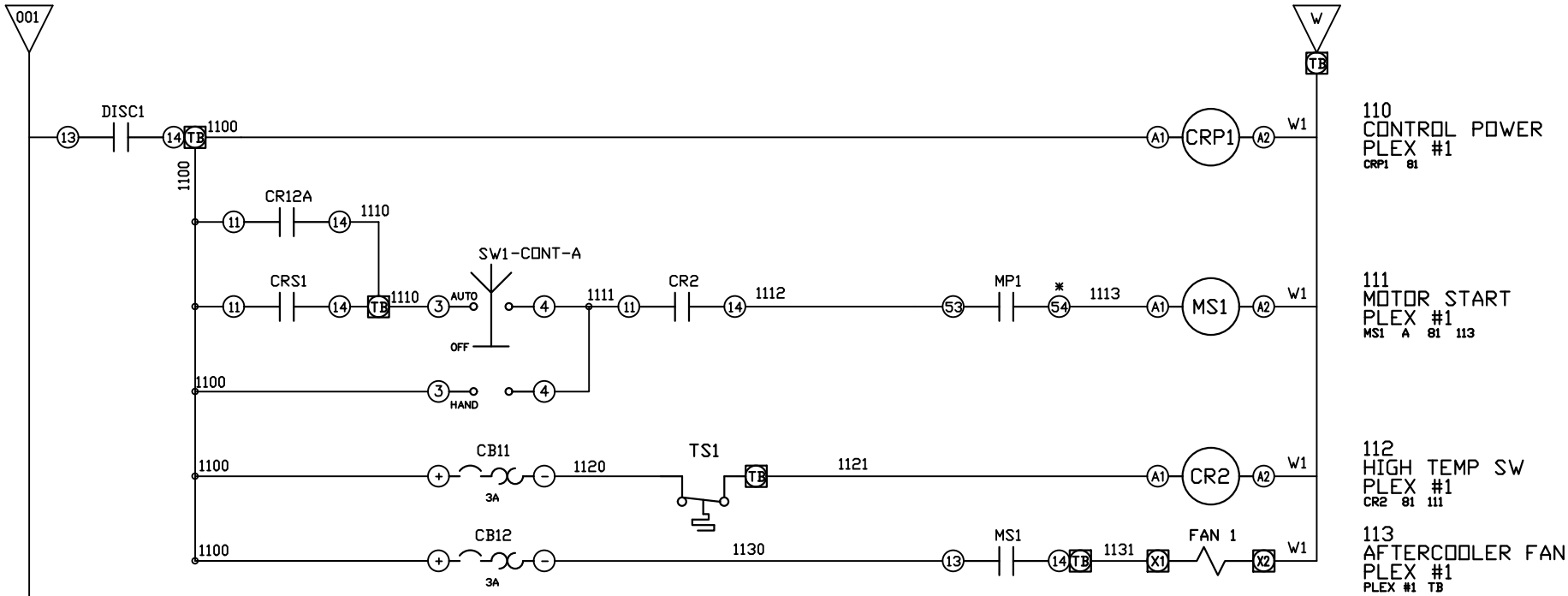

* FOR LAYOUT-B MOTOR PROTECTORS
USE PINS #1 AND #4 ON AUX CONTACTS

OPTIONAL:

CONFIDENTIAL DISCLOSURE:
This drawing is the property of POWEREX, INC. of Marmon Specialty Products and subject to return on demand. Its contents are confidential and must not be copied or submitted to outside parties for use or examination.

DRAWN BY	CHECKED BY	ENGINEERING APPROVAL
CHR	BFH	DMS
07/20/11	03/12/12	03/12/12


PANEL TYPE
MED AIR TRIPLEX
HMI, NFPA

DWG. TYPE
WIRING

DWG. NO.
PXMI-A316 W

SHEET W-3	SIZE B
--------------	-----------

PXMI-A316 B1

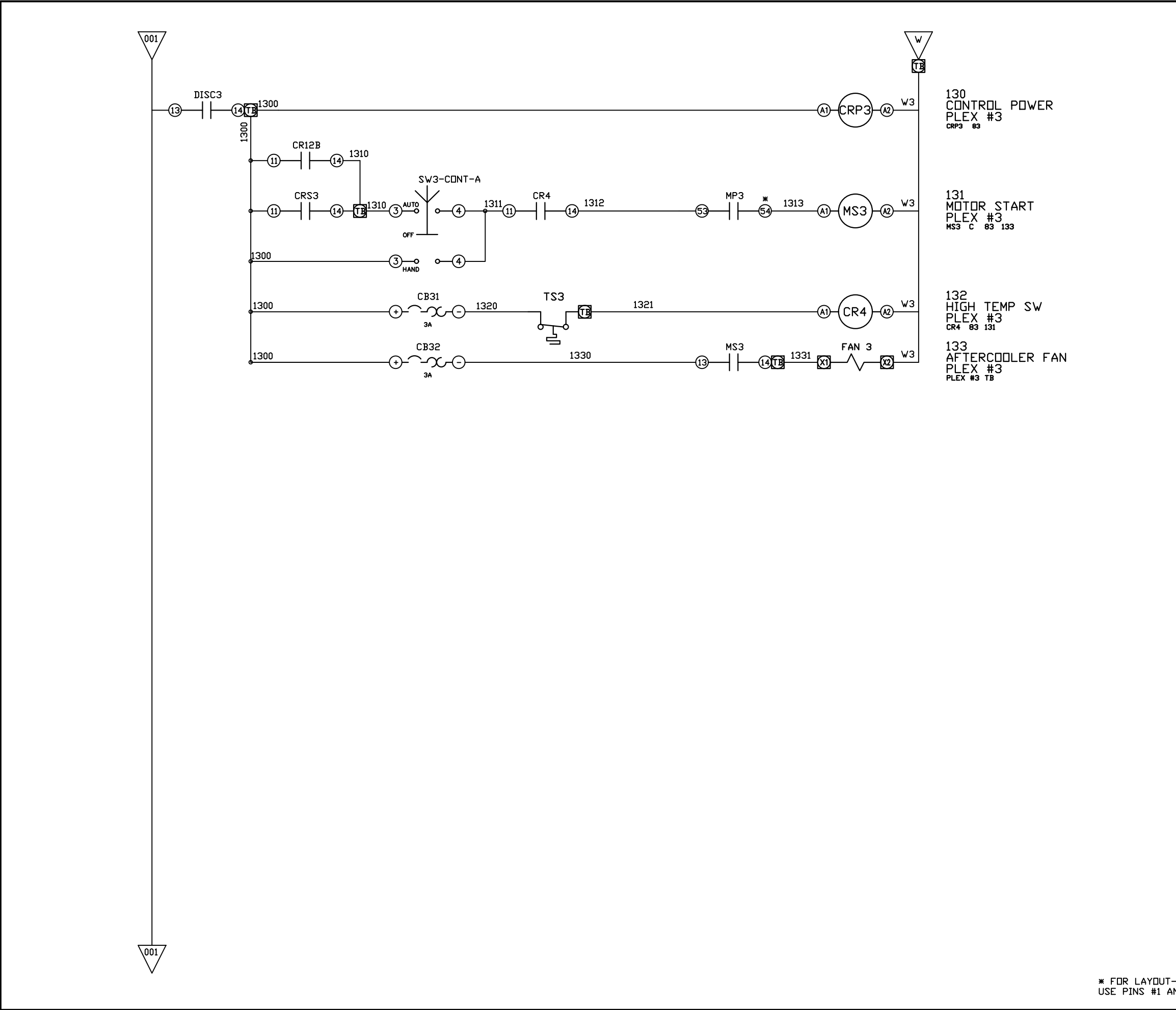

REV	REVISION	DATE	ECN	NAME	CHKD
B-1	RELEASE TO PRODUCTION	07/20/11	22545	CHR	CHR
G-1	ADDED GROUND TERMINAL BLOCKS	04/11/25	PXEC0688	JNW	JRD
H-1	ADDED PLEX ISOLATION	05/29/25	PXEC0639	SEK	WMM

OPTIONAL:



CONFIDENTIAL DISCLOSURE:
This drawing is the property of POWEREX, INC. of Marmon Specialty Products and subject to return on demand. Its contents are confidential and must not be copied or submitted to outside parties for use or examination.

DRAWN BY	CHECKED BY	ENGINEERING APPROVAL
CHR	BFH	DMS
07/20/11	03/12/12	03/12/12

PANEL TYPE
MED AIR TRIPLEX
HMI, NFPA



DWG. TYPE WIRING

DWG. NO.
PXMI-A316 W

SHEET	SIZE
W-4	B

* FOR LAYOUT-B MOTOR PROTECTORS
USE PINS #1 AND #4 ON AUX CONTACTS

REV	REVISION	DATE	ECN	NAME	CHKD
B-1	RELEASE TO PRODUCTION	07/20/11	22545	CHR	CHR
G-1	ADDED GROUND TERMINAL BLOCKS	04/11/25	PXEC0688	JNW	JRD
H-1	ADDED PLEX ISOLATION	05/29/25	PXEC0639	SEK	WMM

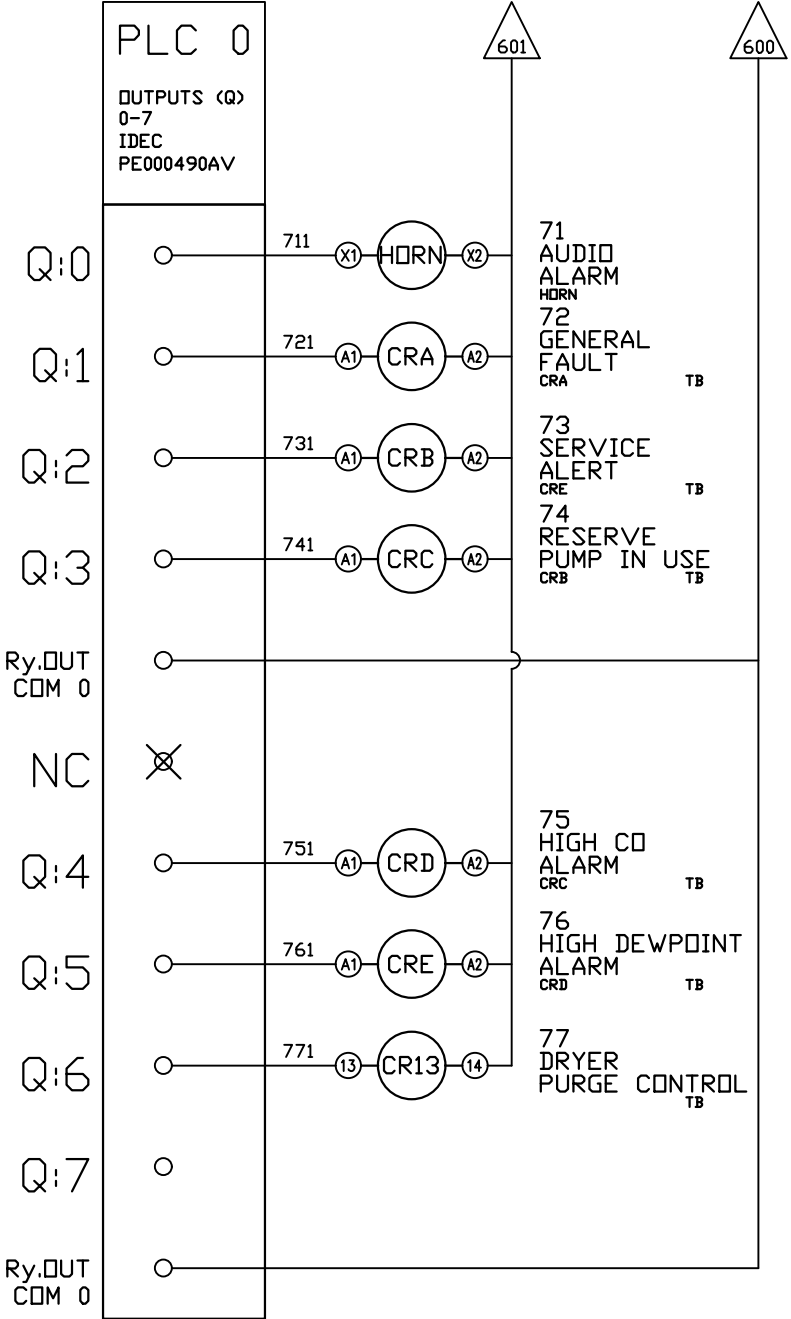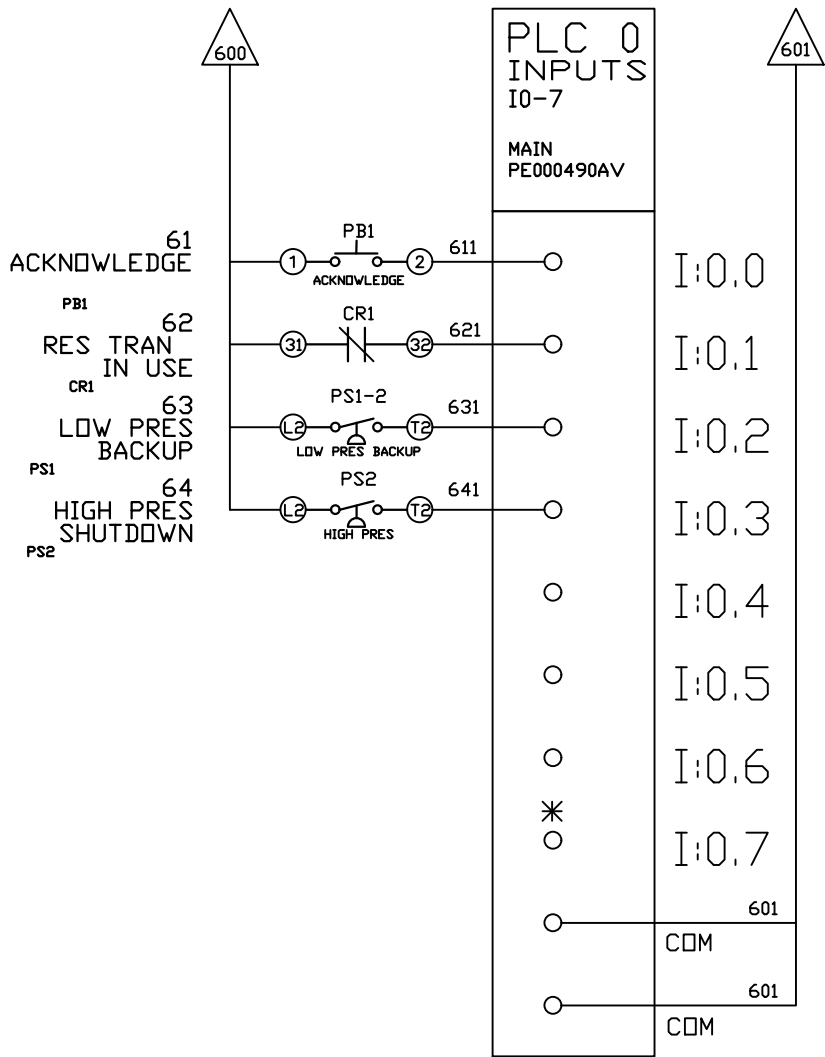

OPTIONAL:

POWEREX™

CONFIDENTIAL DISCLOSURE:
This drawing is the property of POWEREX, INC. of Marmon Specialty Products and subject to return on demand. Its contents are confidential and must not be copied or submitted to outside parties for use or examination.

DRAWN BY	CHECKED BY	ENGINEERING APPROVAL
CHR	BFH	DMS
07/20/11	03/12/12	03/12/12

PANEL TYPE
MED AIR TRIPLEX
HMI, NFPA


DWG. TYPE WIRING

DWG. NO.
PXMI-A316 W

SHEET	SIZE
W-6	B

*FACTORY TEST JUMPER LOCATION

REV	REVISION	DATE	ECN	NAME	CHKD
B-1	RELEASE TO PRODUCTION	07/20/11	22545	CHR	CHR
G-1	ADDED GROUND TERMINAL BLOCKS	04/11/25	PXEC0688	JNW	JRD
H-1	ADDED PLEX ISOLATION	05/29/25	PXEC0639	SEK	WMM

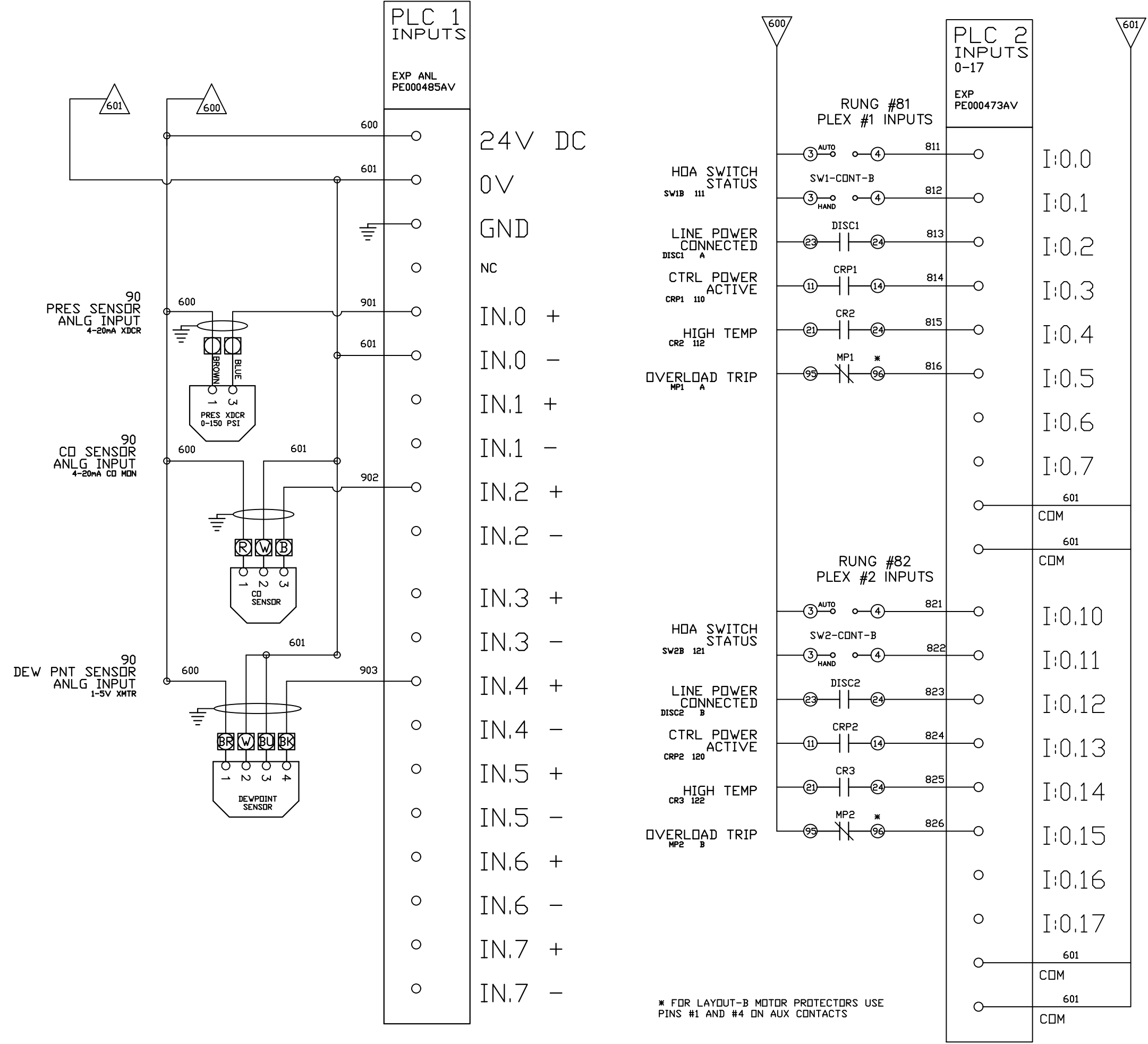

OPTIONAL:



CONFIDENTIAL DISCLOSURE:
This drawing is the property of POWEREX, INC. of Marmon Specialty Products and subject to return on demand. Its contents are confidential and must not be copied or submitted to outside parties for use or examination.



DRAWN BY	CHECKED BY	ENGINEERING APPROVAL
CHR	BFH	DMS
07/20/11	03/12/12	03/12/12

PANEL TYPE
MED AIR TRIPLEX
HMI, NFPA

DWG. TYPE
WIRING

DWG. NO.
PXMI-A316 W

SHEET	SIZE
W-7	B

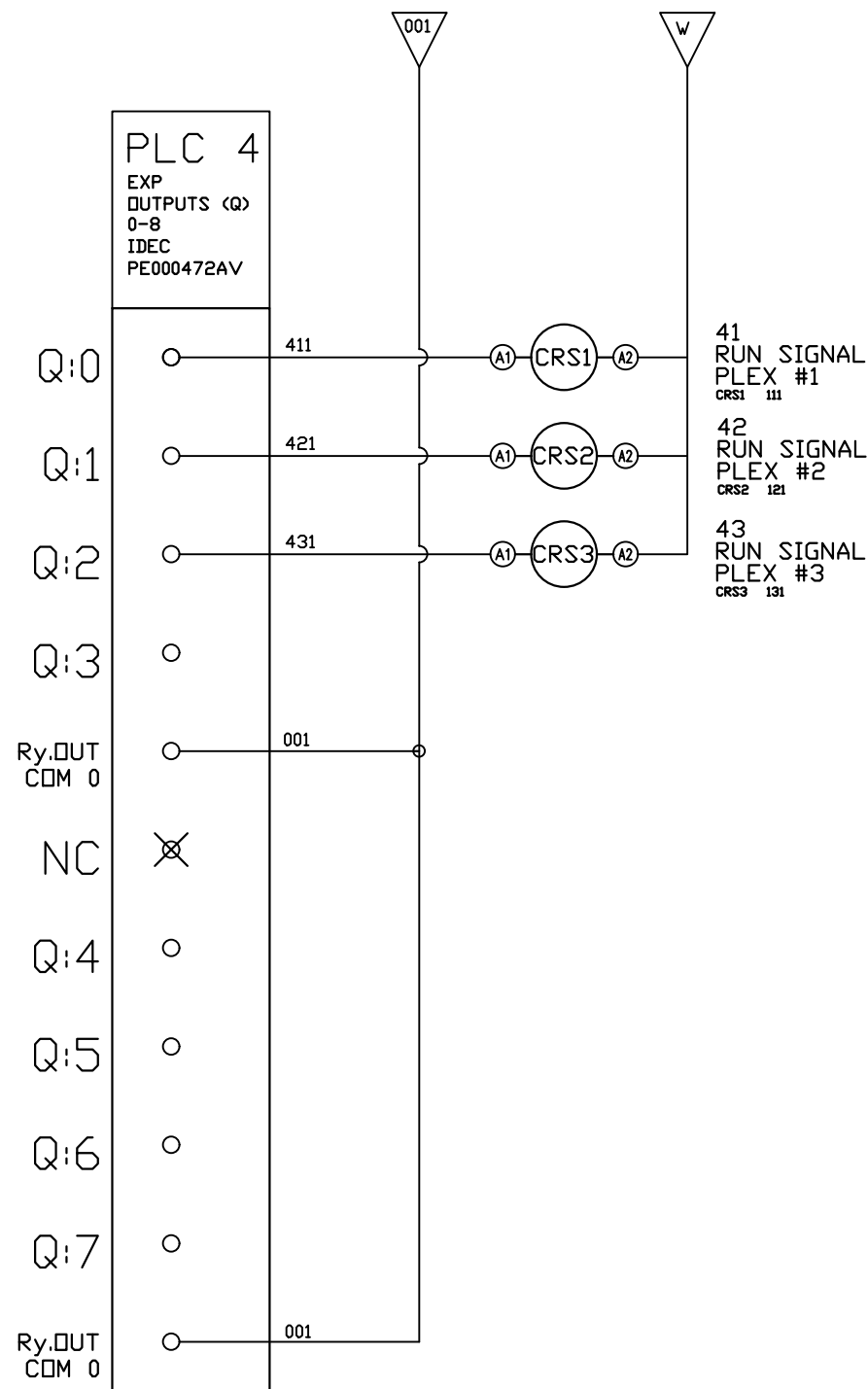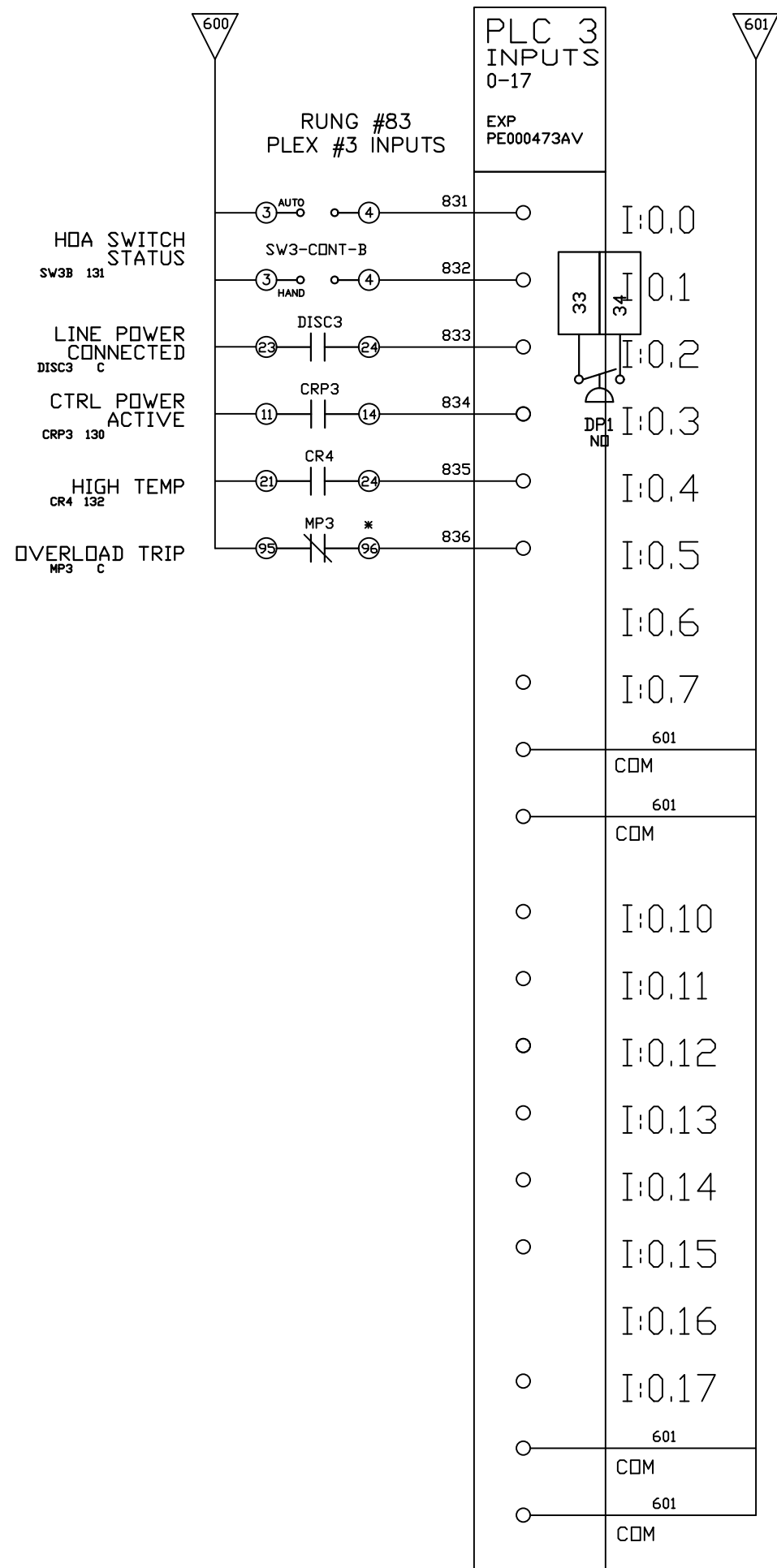


REV	REVISION	DATE	ECN	NAME	CHKD
B-1	RELEASE TO PRODUCTION	07/20/11	22545	CHR	CHR
G-1	ADDED GROUND TERMINAL BLOCKS	04/11/25	PXEC0688	JNW	JRD
H-1	ADDED PLEX ISOLATION	05/29/25	PXEC0639	SEK	WMM


OPTIONAL:		
		
CONFIDENTIAL DISCLOSURE: This drawing is the property of POWEREX, INC. of Marmon Specialty Products and subject to return on demand. Its contents are confidential and must not be copied or submitted to outside parties for use or examination.		
DRAWN BY	CHECKED BY	ENGINEERING APPROVAL
CHR	BFH	DMS
07/20/11	03/12/12	03/12/12
PANEL TYPE		
MED AIR TRIPLEX HMI, NFPA		
DWG. TYPE WIRING		
DWG. NO.		
PXMI-A316 W		
SHEET		SIZE
W-8		B
PXMI-A316 B1		

TERMINAL BLOCKS


REV	REVISION	DATE	ECN	NAME	CHKD
B-1	RELEASE TO PRODUCTION	07/20/11	22545	CHR	CHR
G-1	ADDED GROUND TERMINAL BLOCKS	04/11/25	PXEC0688	JNW	JRD
H-1	ADDED PLEX ISOLATION	05/29/25	PXEC0639	SEK	WMM

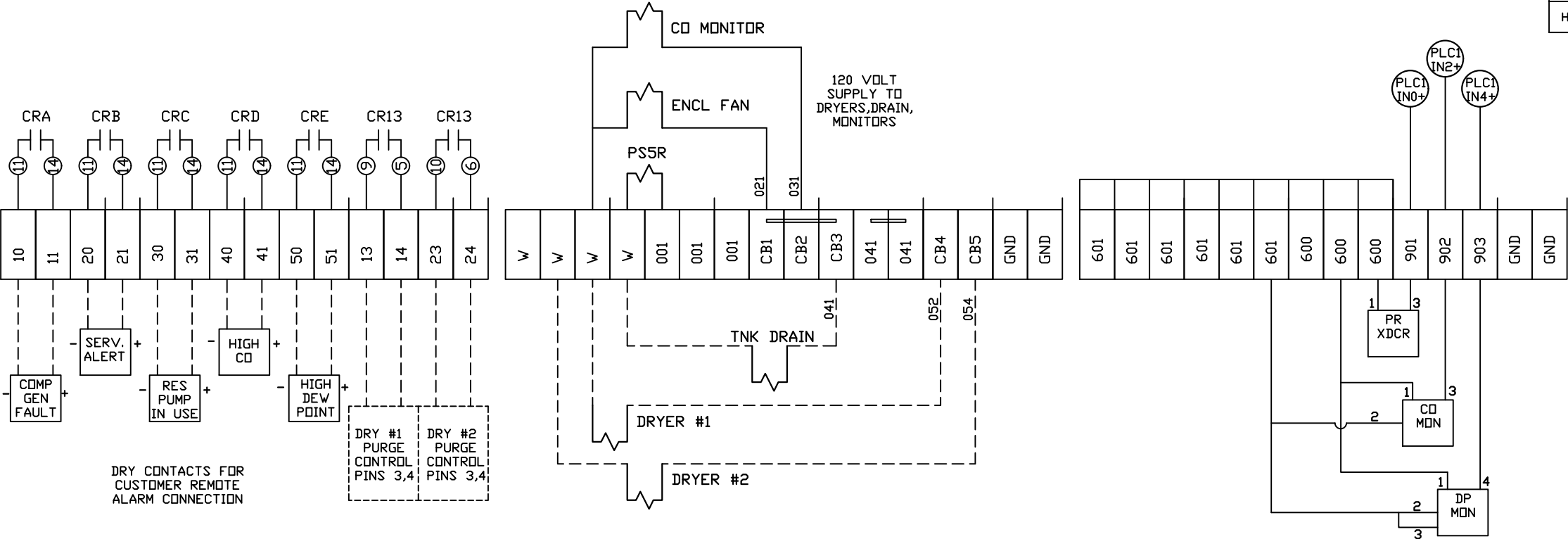


PLEX #1 CONNECTIONS

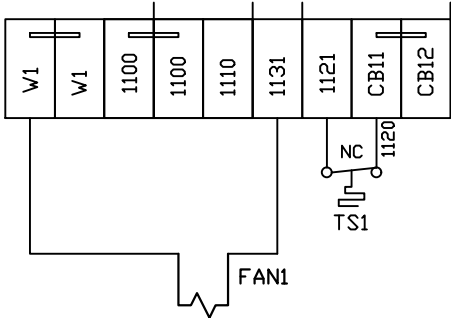

PLEX #2 CONNECTIONS

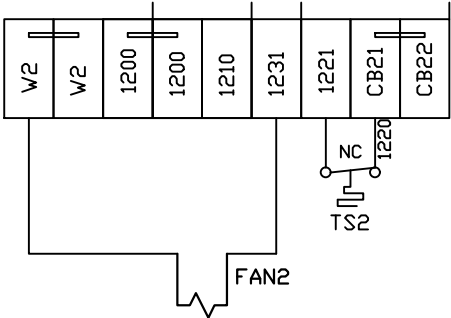

PLEX #3 CONNECTIONS

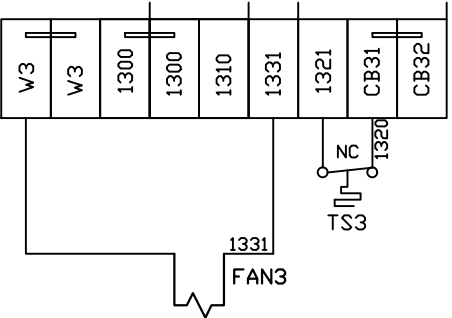

FIELD WIRING NOTES:

- PANEL GROUND MUST BE CONNECTED TO EARTH GROUND
- TRANSFORMER IS SIZED FOR LOADS SHOWN ON DRAWING ONLY.
- FIELD WIRING INDICATED BY -----, WIRE TO BE COPPER RATED AT 75° C.
- ALL ALARM DRY CONTACTS ARE CLOSED FOR NORMAL OPERATION AND OPEN IN ALARM.
- DRY CONTACT RATING: 30V DC/2A (resistive load, inductive load L/R = 7 ms)
- USE CAT-5 CABLE W/RJ45 CONNECTOR FOR ETHERNET CONNECTION TO COMMUNICATION NETWORK OR BAS SYSTEM

OPTIONAL:

POWEREX™

CONFIDENTIAL DISCLOSURE:
This drawing is the property of POWEREX, INC. of Marmon Specialty Products and subject to return on demand. Its contents are confidential and must not be copied or submitted to outside parties for use or examination.

DRAWN BY	CHECKED BY	ENGINEERING APPROVAL
CHR	BFH	DMS
07/20/11	03/12/12	03/12/12


PANEL TYPE
MED AIR TRIPLEX
HMI, NFPA

DWG. TYPE
WIRING

DWG. NO.
PXMI-A316 W

SHEET	SIZE
W-9	B

REV	REVISION	DATE	ECN	NAME	CHKD
B-1	RELEASE TO PRODUCTION	07/20/11	22545	CHR	CHR
G-1	ADDED GROUND TERMINAL BLOCKS	04/11/25	PXEC0688	JNW	JRD
H-1	ADDED PLEX ISOLATION	05/29/25	PXEC0639	SEK	WMM

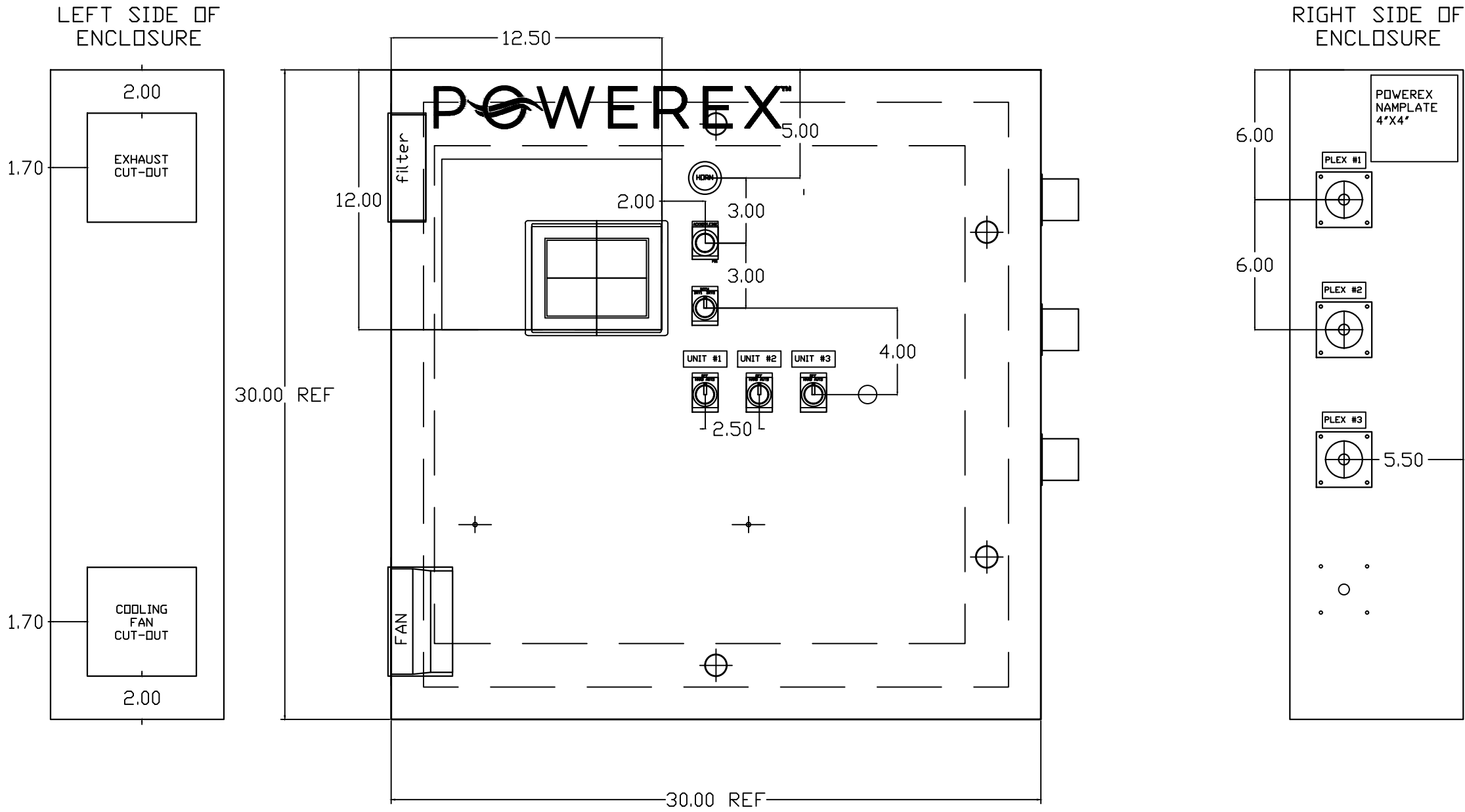

OPTIONAL:

CONFIDENTIAL DISCLOSURE:

This drawing is the property of POWEREX, INC. of Marmon Specialty Products and subject to return on demand. Its contents are confidential and must not be copied or submitted to outside parties for use or examination.

DRAWN BY	CHECKED BY	ENGINEERING APPROVAL
CHR	BFH	DMS
07/20/11	03/12/12	03/12/12

PANEL TYPE

MED AIR TRIPLEX
HMI, NFPA


DWG. TYPE LAYOUT

DWG. NO. PXMI-A316 W

SHEET	SIZE
LA-1	B

PXMI-A316 B1

REV	REVISION	DATE	ECN	NAME	CHKD
B-1	RELEASE TO PRODUCTION	07/20/11	22545	CHR	CHR
G-1	ADDED GROUND TERMINAL BLOCKS	04/11/25	PXEC0688	JNW	JRD
H-1	ADDED PLEX ISOLATION	05/29/25	PXEC0639	SEK	WMM

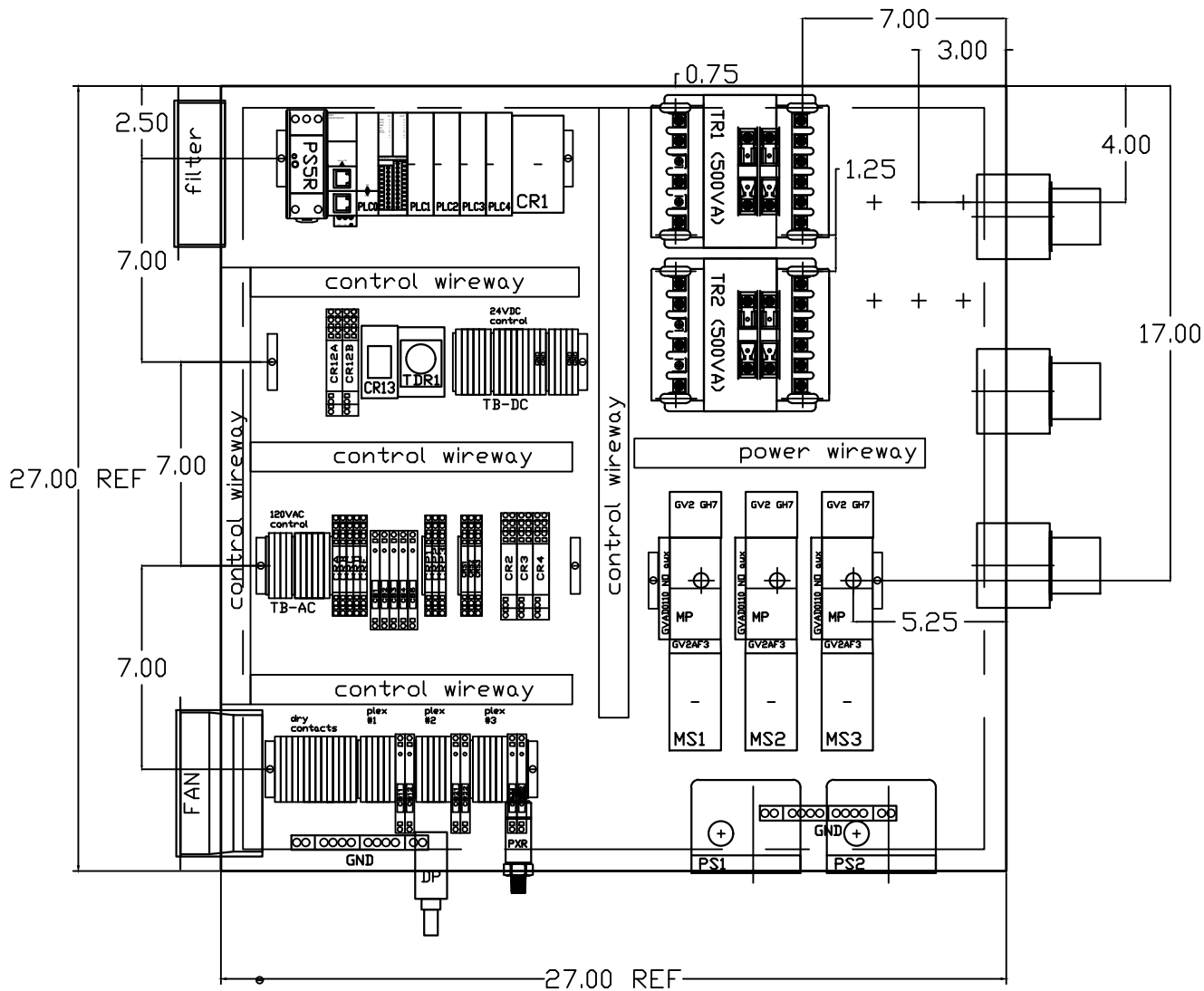

OPTIONAL:

POWEREX™

CONFIDENTIAL DISCLOSURE:
This drawing is the property of POWEREX, INC. of Marmon Specialty Products and subject to return on demand. Its contents are confidential and must not be copied or submitted to outside parties for use or examination.

DRAWN BY	CHECKED BY	ENGINEERING APPROVAL
CHR	BFH	DMS
07/20/11	03/12/12	03/12/12


PANEL TYPE
MED AIR TRIPLEX
HMI, NFPA

DWG. TYPE LAYOUT

DWG. NO.
PXMI-A316 W

SHEET	SIZE
LA-2	B

REV	REVISION	DATE	ECN	NAME	CHKD
B-1	RELEASE TO PRODUCTION	07/20/11	22545	CHR	CHR
G-1	ADDED GROUND TERMINAL BLOCKS	04/11/25	PXEC0688	JNW	JRD
H-1	ADDED PLEX ISOLATION	05/29/25	PXEC0639	SEK	WMM

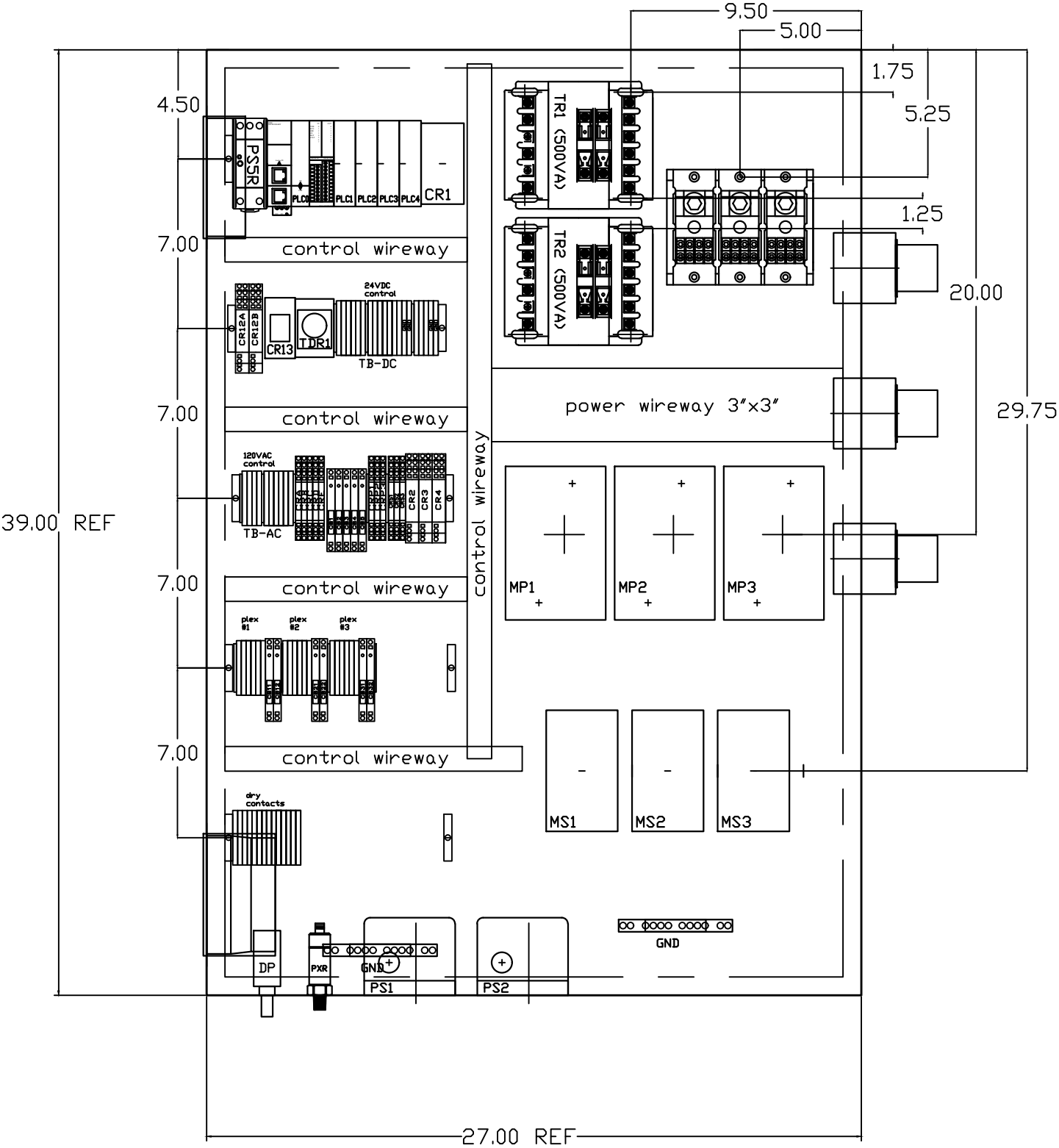

OPTIONAL:

CONFIDENTIAL DISCLOSURE:
This drawing is the property of POWEREX, INC. of Marmon Specialty Products and subject to return on demand. Its contents are confidential and must not be copied or submitted to outside parties for use or examination.

DRAWN BY	CHECKED BY	ENGINEERING APPROVAL
CHR	BFH	DMS
07/20/11	03/12/12	03/12/12

PANEL TYPE

MED AIR TRIPLEX
HMI, NFPA


DWG. TYPE LAYOUT

DWG. NO. PXMI-A316 W

SHEET	SIZE
LB-2	B

PXMI-A316 B1

REV	REVISION	DATE	ECN	NAME	CHKD
B-1	RELEASE TO PRODUCTION	07/20/11	22545	CHR	CHR
G-1	ADDED GROUND TERMINAL BLOCKS	04/11/25	PXEC0688	JNW	JRD
H-1	ADDED PLEX ISOLATION	05/29/25	PXEC0639	SEK	WMM

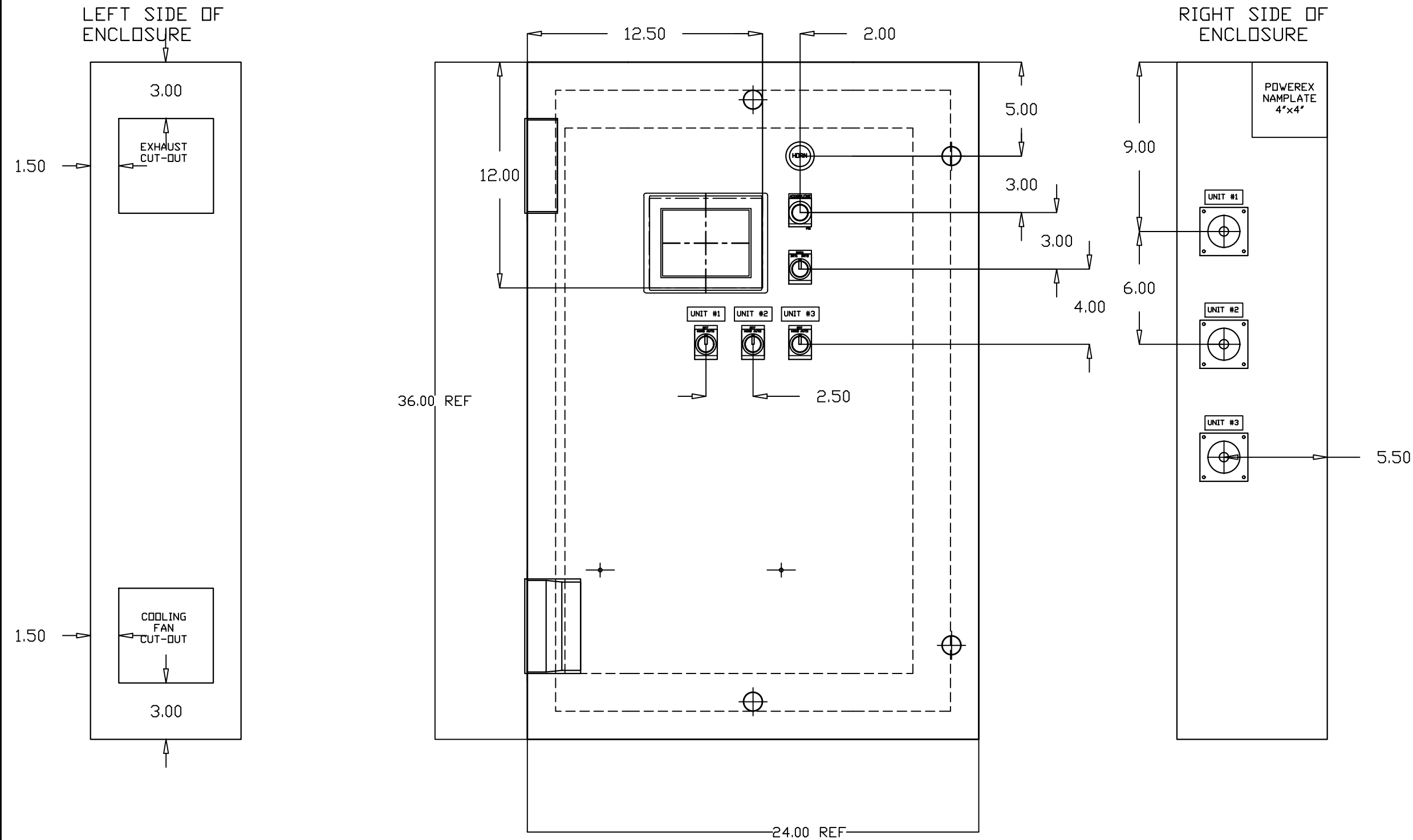

OPTIONAL:

CONFIDENTIAL DISCLOSURE:

This drawing is the property of POWEREX, INC. of Marmon Specialty Products and subject to return on demand. Its contents are confidential and must not be copied or submitted to outside parties for use or examination.

DRAWN BY	CHECKED BY	ENGINEERING APPROVAL
CHR	BFH	DMS
07/20/11	03/12/12	03/12/12

PANEL TYPE

MED AIR TRIPLEX
HMI, NFPA


DWG. TYPE LAYOUT

DWG. NO. PXMI-A316 W

SHEET	SIZE
LC-1	B

PXMI-A316 B1

REV	REVISION	DATE	ECN	NAME	CHKD
B-1	RELEASE TO PRODUCTION	07/20/11	22545	CHR	CHR
G-1	ADDED GROUND TERMINAL BLOCKS	04/11/25	PXEC0688	JNW	JRD
H-1	ADDED PLEX ISOLATION	05/29/25	PXEC0639	SEK	WMM

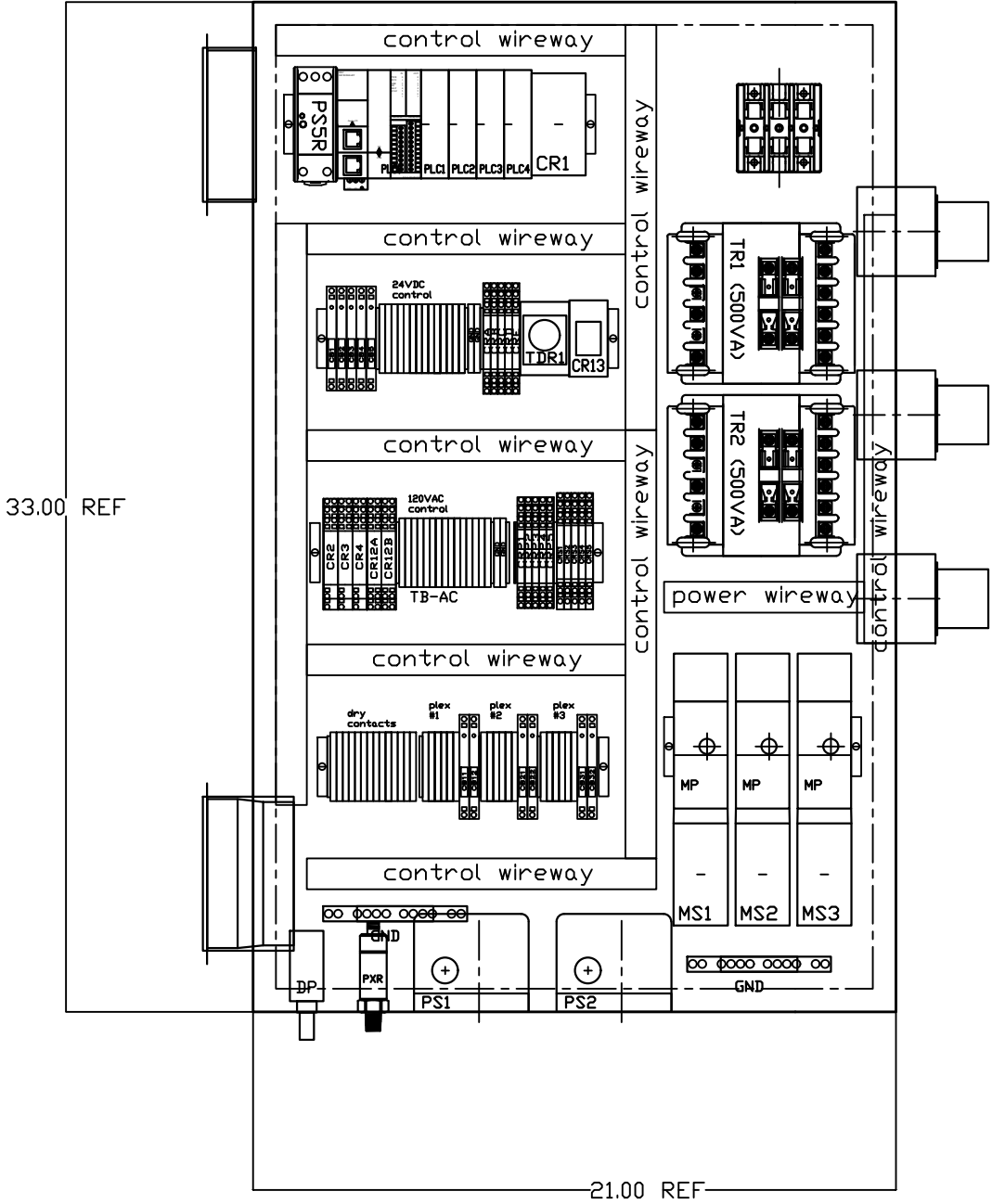

OPTIONAL:

CONFIDENTIAL DISCLOSURE:
This drawing is the property of POWEREX, INC. of Marmon Specialty Products and subject to return on demand. Its contents are confidential and must not be copied or submitted to outside parties for use or examination.

DRAWN BY	CHECKED BY	ENGINEERING APPROVAL
CHR	BFH	DMS
07/20/11	03/12/12	03/12/12

PANEL TYPE

MED AIR TRIPLEX
HMI, NFPA

DWG. TYPE LAYOUT

DWG. NO. PXMI-A316 W

SHEET	SIZE
LC-2	B

PXMI-A316 B1

TABLE 1	P/N	MOTOR FULL LOAD AMPS	TOTAL FULL LOAD AMPS	NON-TIME DELAY FUSE	TIME DELAY FUSE	INVERSE- TIME CIRCUIT BREAKER
208V (3Ø)	02AJ	4.1	14.3	25	20	25
	12AJ	4.5	15.5	25	20	25
	22AJ	5.6	18.8	30	25	30
	32AJ	8.8	28.4	50	35	45
	52AJ	14.6	45.8	80	60	70
	72AJ	20.5	63.5	110	80	100
	A2AJ *	27.4	84.2	150	125	150
	F2AJ *	41.1	125.3	225	175	200
230V (3Ø)	03AJ	3.6	12.8	20	20	20
	13AJ	4.4	15.2	25	20	25
	23AJ	5.8	19.4	35	25	30
	33AJ	7.7	25.1	45	35	40
	53AJ	12.7	40.1	70	50	60
	73AJ	18.5	57.5	100	80	90
	A3AJ	24.8	76.4	150	100	125
	F3AJ *	37.2	113.6	200	150	175
460V (3Ø)	04AJ	1.8	7.4	15	10	15
	14AJ	2.2	8.6	15	10	15
	24AJ	2.7	10.1	20	15	15
	34AJ	3.9	13.7	25	20	25
	54AJ	6.3	20.9	35	30	35
	74AJ	9.3	29.9	50	40	45
	A4AJ	12.4	39.2	70	50	60
	F4AJ	18.6	57.8	100	80	90
380V (3Ø) 50HZ	08AJ					
	18AJ					
	28AJ	3.4	12.2	20	15	20
	38AJ	4.6	15.8	25	20	25
	58AJ	7.7	25.1	45	35	40
	78AJ	11.1	35.3	60	45	60
	A8AJ	14.9	46.7	80	60	70
575V (3Ø)	F8AJ	22.1	68.3	125	90	125
	07AJ	1.1	5.3	10	10	10
	17AJ					
	27AJ	2.3	8.9	15	15	15
	37AJ	3.1	11.3	20	15	20
	57AJ	5.1	17.3	30	25	30
	77AJ	6.9	22.7	40	30	35
	A7AJ	9.9	31.7	60	40	50
	F7AJ	14.9	46.7	80	60	70

- NOTES:
1. RECOMMENDED TIGHTENING TORQUES FOR WIRE TERMINALS:
208-575 VOLT POWER 35 POUND INCHES
120 VOLT POWER AND CONTROL VOLTAGE 15 POUND INCHES

2. PANEL GROUND MUST BE CONNECTED TO EARTH GROUND

3. INSTALLER TO PROVIDE MAIN DISCONNECTING DEVICE FOR THIS ELECTRICAL ASSEMBLY. SHORT CIRCUIT PROTECTION IS RECOMMENDED, SEE TABLE 1. IF PROVIDING PANEL SHORT CIRCUIT PROTECTION USING TIME DELAY FUSES, PANEL SCCR IS 65kA. OTHERWISE, PANEL SCCR IS 5kA.

4. ALL WIRES MUST BE LABELED ON BOTH ENDS

5. TRANSFORMER IS SIZED FOR LOADS SHOWN ON DRAWING ONLY. DO NOT CONNECT ANY OTHER DEVICES

6. △ -INDICATES A DRAWING WIRE CONNECTION TO ANOTHER PAGE.

7. SERVICE DISCONNECT USAGE - turn HOA switch to OFF position. Turn disconnect OF SAME UNIT to OFF position. Associated PLEX will be isolated for maintenance purposes. After completing maintenance, turn disconnect to ON position and return HOA switch to AUTO. This will return PLEX to normal operation.

CONTROL TRANSFORMERS - 500VA						WIRE TYPE TABLE			
REPLACEMENT FUSE TYPE		208	230	460 380	575				
		VOLT	VOLT	VOLT	VOLT	VOLTAGE	WIRE NUMBERS	GAUGE	COLOR
FU1,2A	FNQR	6A	5A	5A	4A	120VAC	001-599,1100+	16-18AWG	RED/BLK
FU1,2B	(CC)					0VAC	W	16-18AWG	WHT/BLK
						24VDC	600-1099	16-18AWG	PURPLE
FU3A,B	FNM	7A	7A	7A	7A	0VDC	601	16-18AWG	PURPLE
						GND	-	VARIES	GREEN
						CUSTOMER	01-99	16-18AWG	YELLOW
SEE XFRMR FOR CONNECTIONS									

SEQUENCE OF OPERATIONS

During normal operation the PBMI controller will signal the Lead compressor to run when pressure drops below lead cut-in set-point and stop when the pressure reaches the lead cut-out set-point. Lead alternation to the next pump, will occur with each lead run signal or every 10-minutes (which ever happens first). If demand cannot be satisfied by the lead pump, the lag pump(s) will start and stop based upon the lag cut-in and cut-out set-points. When more than one pump is running, lead alternation will occur when the lowest cut-out set-point is satisfied, or after 10-minutes (which ever happens first). The HOA switch’s place the pump in the following modes: Hand-turns pump on to run continuous. Off-disables pump from running. Auto-places pump in the “ready mode” and will start and stop based on sequence described above.

All plex configurations include a hardwired Back-up pressure switch circuit should a control failure occur. This circuit will call all pumps on and off based on the reserve pressure switch set-points.

Expandable systems include all control devices, operators, and programming for the maximum number of pumps (or plex) required. To expand the system: navigate to the “service screen” and enter the number of pumps.

Additional information and descriptions can be accessed through the HMI “service info” screen by pressing Sequence of Operations button.

REV	REVISION	DATE	ECN	NAME	CHKD
B-1	RELEASE TO PRODUCTION	03/12/12	22545	BFH	CHR
C-1	ADDED 380/575V	02/26/13	PXEC0030	BFH	CHR
D-1	FORMAT UPDATE	03/04/13	PXEC0038	BFH	CHR

OPTIONAL:

208V-575V



CONFIDENTIAL DISCLOSURE:
This drawing is the property of POWEREX INC of the SCOTT FETZER COMPANY and subject to return on demand. Its contents are confidential and must not be copied or submitted to outside parties for use or examination.

DRAWN BY
BFH

CHECKED BY
CHR

ENGINEERING APPROVAL
CHR

03/12/12

03/12/12

03/12/12

PANEL TYPE

COMPRESSOR DATA TABLE
3PH TRIPLEX PANEL DATA

DWG. TYPE
MISC

DWG. NO.
DATA TABLE

SHEET
D-1

SIZE
B

TRIPLEX PANEL DATA

* - USE LAYOUT B